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THE SPECIFICATIONS AND INFORMATION REGARDING THE PRODUCTS IN THIS MANUAL ARE SUBJECT TO CHANGE WITHOUT NOTICE. ALL STATEMENTS, INFORMATION, AND RECOMMENDATIONS IN THIS MANUAL ARE BELIEVED TO BE ACCURATE BUT ARE PRESENTED WITHOUT WARRANTY OF ANY KIND, EXPRESS OR IMPLIED. USERS MUST TAKE FULL RESPONSIBILITY FOR THEIR APPLICATION OF ANY PRODUCTS.

THE SOFTWARE LICENSE AND LIMITED WARRANTY FOR THE ACCOMPANYING PRODUCT ARE SET FORTH IN THE INFORMATION PACKET THAT SHIPPED WITH THE PRODUCT AND ARE INCORPORATED HEREIN BY THIS REFERENCE. IF YOU ARE UNABLE TO LOCATE THE SOFTWARE LICENSE OR LIMITED WARRANTY, CONTACT YOUR CISCO REPRESENTATIVE FOR A COPY.

The following information is for FCC compliance of Class A devices: This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio-frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference, in which case users will be required to correct the interference at their own expense.

Modifications to this product not authorized by Cisco could void the FCC approval and negate your authority to operate the product.

NOTWITHSTANDING ANY OTHER WARRANTY HEREIN, ALL DOCUMENT FILES AND SOFTWARE OF THESE SUPPLIERS ARE PROVIDED "AS IS" WITH ALL FAULTS. CISCO AND THE ABOVE-NAMED SUPPLIERS DISCLAIM ALL WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING, WITHOUT LIMITATION, THOSE OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NON INFRINGEMENT OR ARISING FROM A COURSE OF DEALING, USAGE, OR TRADE PRACTICE.

IN NO EVENT SHALL CISCO OR ITS SUPPLIERS BE LIABLE FOR ANY INDIRECT, SPECIAL, CONSEQUENTIAL, OR INCIDENTAL DAMAGES, INCLUDING, WITHOUT LIMITATION, LOST PROFITS OR LOSS OR DAMAGE TO DATA ARISING OUT OF THE USE OR INABILITY TO USE THIS MANUAL, EVEN IF CISCO OR ITS SUPPLIERS HAVE BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES.

All trademarks mentioned in this document or website are the property of their respective owners. The use of the word partner does not imply a partnership relationship between Cisco and any other company. (0910R)

Cisco Systems Inc.
170 West Tasman Dr.
San Jose, Ca 95134 USA

Emergo Europe
Molenstraat 15
2513 BH
The Hague, Netherlands
+31.70.345.8570 - phone
+31.70.346.7299 -

Device Components

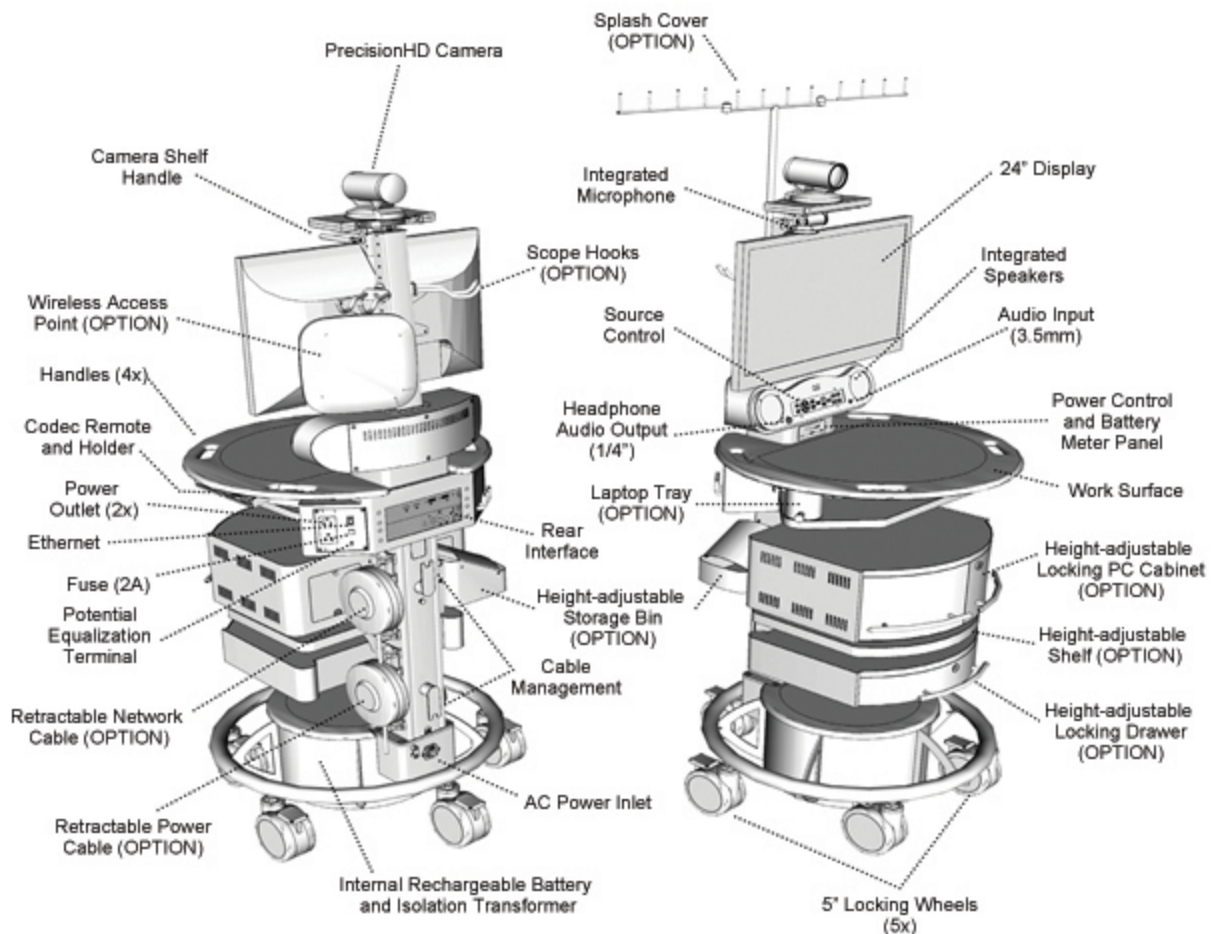
Figure 1.1

The Cisco Telepresence VX Clinical Assistant™ consists of a number of components, as illustrated in Figure 1-1. For more information, please refer to the **Cisco Telepresence VX Clinical Assistant™ (VXCA)** Installation and User Guide at http://www.cisco.com/en/US/products/ps12152/tsd_products_support_series_home.html

Intended Use

The Cisco TelePresence VX Clinical Assistant™ (VXCA) enables secure and effective audio/video communication between multiple healthcare providers and between providers and patients. The system transmits real-time audio and video captured by a high-definition camera and integrated microphone. It also receives real-time audio and video from a corresponding functionally like system in a remote location. Finally, the VXCA supports transmission of real-time audio and video signals from peripheral audio/visual medical devices, which may be connected to standard audio and video inputs provided on the system.

The VXCA is intended to facilitate remote provider/patient and provider/provider consultations. These consultations should always be conducted with a licensed medical professional physically in the room with the patient.

Contraindications

The VXCA is not intended to substitute for the in-person physical examination of a patient or as a substitute for direct medical intervention. It is also not intended for real-time, active, or online patient monitoring, nor is it intended to provide time sensitive data or alarms. The VXCA does not support transmission of numerical telemetric/serial data and is not intended for use with non-audio/visual medical devices.

If the user is unable to use the Clinical Assistant for its intended use, discontinue the patient examination using the VXCA and refer to the Cisco TelePresence VX Clinical Assistant™ Installation and User Guide for further instructions on proper operation. If a problem cannot be resolved by the user, contact Cisco Systems for further support at http://www.cisco.com/en/US/support/tsd_cisco_worldwide_contacts.html.

System Considerations

The user's infrastructure must meet basic requirements in order for VXCA to perform as intended. This includes, but is not limited to: adequate bandwidth, appropriate capacity routing and switching equipment. The VXCA should only be used with connected peripheral devices that are compatible with the audio and video specifications of the VXCA, as set forth in the Installation and User Guide.

Patient Privacy

The Cisco TelePresence VX Clinical Assistant™ employs encryption during the Telepresence call, encryption must be supported by both end points to be active in the call. No security features can provide absolute protection against loss of privacy. The user is responsible for using reasonable precautions at its site to secure appropriate patient confidentiality.

Therapeutic Claims

Cisco does not claim that the Cisco TelePresence VX Clinical Assistant will improve medical outcomes or have a therapeutic impact on patients with specific medical conditions.

Warnings

Not Intended For Dialing Emergency Medical Services

The Cisco TelePresence VX Clinical Assistant is not intended to make emergency calls. In the event of an emergency, the user should use a telephone or device other than the Cisco TelePresence VX Clinical Assistant to call local emergency services.

Loss of Connectivity

During the use of the Cisco TelePresence VX Clinical Assistant™, a healthcare professional may lose connectivity with the system, a medical device or the video conferencing system. Loss of connectivity can result from power outages, network outages, and failure in the VXCA software/hardware, or other causes. Loss of connectivity can prevent a health care professional from completing the patient exam in a timely manner. In cases where performance of the examination is time critical, the user should not use the Cisco TelePresence VX Clinical Assistant System and instead seek an in-person examination from a licensed healthcare professional.

Delay or Choppiness of Audio or Video Transmission

The Cisco TelePresence VX Clinical Assistant normally operates at a delay of no more than 200 milliseconds, meaning that the provider will experience the audio and video transmissions later than they are really occurring. In cases where the remote healthcare provider needs to make time sensitive assessments or give instructions this delay may result in misalignment of audio and visual transmissions to the real-time procedure and/or inappropriate treatment or diagnosis.

Additional Latency Information

Excess latency in any Cisco Telepresence meeting degrades the in-person experience. Latency becomes an even larger issue with multipoint calls because all CTS systems dial in to a CTMS that might not be located in the same geographical location as the CTS endpoints. Due to the nature of multipoint, two CTS endpoints that provide low latency in a point to point meeting might have considerably higher latency in a multipoint meeting. Inserting and multipoint device in the media path of a Telepresence call introduces additional latency. However, proper placement of the CTMS helps minimize latency and preserve the Cisco Telepresence experience.

A Cisco Telepresence network should always be designed to target own-way, end to end, network latency less than 150ms. However, in some cases this is not possible due to long distances between international sites. Therefore, the recommended upper limit allowed for one way, end to end network latency is less than 200 ms.

Proper Training Is Required

Healthcare professionals using the Cisco Telepresence VX Clinical Assistant™ should be sufficiently trained and familiar with the VXCA Installation and User Guide. Please refer to the Cisco TelePresence VX Clinical Assistant™ (CA) Installation and User Guide under “Troubleshooting & Frequently Asked Questions,” “Safety Precautions” and “Operator Safety Precaution.” See http://www.cisco.com/en/US/products/ps12152/tsd_products_support_series_home.html

Camera May Not Focus

There may be times when the main VXCA camera or a connected peripheral camera does not focus or loses focus. This problem can result from equipment malfunction, improper use of the camera, improper configuration, or other reasons. An improperly configured or poorly focused camera may reduce the quality of the image. The camera can be maximized on the inputs on frame rate and resolution. Please refer to the Cisco TelePresence System Codec C Series “Getting Started Guide” at

http://www.cisco.com/en/US/products/ps11422/prod_installation_guides_list.html.

Audio Distortions

The VXCA can experience audio distortions due to network latency, software/hardware malfunction, insufficient bandwidth, or for other reasons. If audio is not of sufficient quality for the intended use of the VXCA, please refer to the Cisco TelePresence VX Clinical Assistant™ Installation and User Guide under “Source Controls,” “Technical Specifications,” and “Troubleshooting & Frequently Asked Questions.” See http://www.cisco.com/en/US/products/ps12152/tsd_products_support_series_home.html

Precautions

To help ensure safety and to minimize operational problems, please refer to the Cisco TelePresence VX Clinical Assistant™ (VXCA) Installation and User Guide under “Operator Safety Summary” and “Important Safeguards.” See http://www.cisco.com/en/US/products/ps12152/tsd_products_support_series_home.html

Improper Transport Methods

When transporting the mobile cart over an obstacle greater than 10 mm, it is recommended that 2 people assist in the transport. The preferred method is to back the rear wheels over the threshold and then rotate the VXCA so that the front two wheels are not both elevated off the ground at the same time while the front of the cart is crossing over the threshold. There are handles on the work surface that are designed to assist in moving and will serve as the fixed transport position.

Do not push or pull the unit by the support column. Use the handles found around the work surface.

Follow Instructions for Use

Avoid Use Near Flammable Mixtures: The VXCA is not suitable for use in the presence of flammable mixtures.

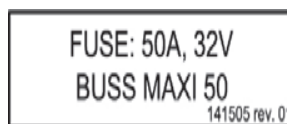
Smash Hazard: Please be aware of areas where there is a potential to smash or crush limbs and extremities or objects.

Power Distribution: The power outlet provided on the system is powered from a medical isolation transformer. The maximum output rating of this outlet is 150 (battery powered) Watts. The “exclamation mark” (see below) within an equilateral triangle is intended to alert the user of important operating and maintenance (servicing) instructions within literature accompanying the equipment.

Potential Equalization Terminal

Environmental: This device is not suitable for use in the presence of a flammable anesthetic mixture with air or oxygen or nitrous oxide. An explosion may result. For more environmental specifications, please refer to the Cisco TelePresence VX Clinical Assistant™ (VXCA) Installation and User Guide under “Technical Specifications.” See http://www.cisco.com/en/US/products/ps12152/tsd_products_support_series_home.html

Compliance: For more safety and EMC specifications, please refer to the Cisco TelePresence VX Clinical Assistant™ (VXCA) Installation and User Guide under “Technical Specifications.” See http://www.cisco.com/en/US/products/ps12152/tsd_products_support_series_home.html

Water and moisture - Do not operate the equipment under or near water - for example near a bathtub, kitchen sink, or laundry tub, or near a swimming pool or in areas with high humidity.

Cleaning - Unplug the apparatus from the wall outlet before cleaning or polishing. Do not use liquid cleaners or aerosol cleaners. Use a lint-free cloth lightly moistened with water for cleaning the exterior of the apparatus.

Ventilation - Do not block any of the ventilation openings of the apparatus. Install in accordance with the installation instructions. Never cover the slots and openings with a cloth or other material. Never install the apparatus near heat sources such as stoves, furnaces, or other apparatus (including amplifiers) that produce heat.

Grounding - This equipment must be grounded. Never defeat the ground conductor or operate the equipment in the absence of a suitably installed ground conductor. Contact the appropriate electrical inspection authority or an electrician if you are uncertain that suitable grounding is available.

Power-Cord Protection - Route the power cord so as to avoid it being walked on or pinched by items placed upon or against it, paying particular attention to the plugs, receptacles, and the point where the cord exits from the apparatus.

Accessories - Use only with accessories specified by the manufacturer, or sold with the apparatus.

Lightning - Unplug this apparatus during lightning storms or when unused for long periods of time.

Servicing - Do not attempt to service the apparatus yourself as opening or removing covers may expose you to dangerous voltages or other hazards, and will void the warranty. Refer all servicing to qualified service personnel.

Storage - If you need to store the system, ensure that it is stored in a controlled environment to avoid damage. Refer to the codec documentation for further guidelines.

Repacking – Do not throw away the carton and packing materials. They make for an ideal container with which to transport the system.

Mobility – Before moving the system unplug the power cord and securely wrap them around the cable wrap brackets. Unplug the table microphone and carry separately. To move the cart, hold the cart handle. You may not use the camera tilt handle for local repositioning of the system. All accessories/additional equipment/laptops should be stored in cabinets or in its proper place before moving the cart.

Damaged Equipment - Unplug the apparatus from the outlet and refer servicing to qualified personnel under the following conditions:

- When the power cord or plug is damaged or frayed;
- If liquid has been spilled or objects have fallen into the apparatus;
- If the apparatus has been exposed to rain or moisture;
- If the apparatus has been subjected to excessive shock by being dropped, or the cabinet has been damaged; or,
- If the apparatus fails to operate in accordance with the operating instructions.

Additional Information

Operation of the VXCA:

For more information on system setup and operation, please refer to the Cisco TelePresence VX Clinical Assistant™ (VXCA) Installation and User Guide: http://www.cisco.com/en/US/products/ps12152/tsd_products_support_series_home.html

To Provide Power to the System:

Plug the system into a standard 120V or 240V outlet as applicable and flip power switch “on” at the bottom of the unit next to the power cord and select the main power button on the power and battery meter panel.

The VXCA is now ready to be connected to the network on the rear of the VXCA, configured and then to begin making calls.

1. **To place or receive a video call**, use the appropriate buttons on the Infrared Remote Control included with the system and follow the on screen prompting.
2. **To connect to other equipment**, there are multiple audio and video inputs on the side, rear, and front interface panels of the VXCA. Devices connected to these inputs can be activated via the front button panel or the included infrared controller.
3. **For more details**, please refer to the C-Series codec documentation:
http://www.cisco.com/en/US/docs/telepresence/endpoint/codec-c-series/tc4/getting_started_guide/profile-series_codec-c-series_quick-set-c20_getting_started_guide_tc41.pdf.

General Cleaning:

1. Before cleaning, ensure that the system is turned off.
2. Direct the suction nozzle of vacuum cleaner to remove dirt/dust accumulated in crevices. Hand brush may be used to assist in removing settled dirt/dust.
3. Use cloth dampens with ethyl alcohol to wipe clean the surfaces. Do not use excessive alcohol as the disinfectant is flammable.
4. Use 0.1% hypochlorite solution to wipe clean the surface of the monitor casings. Wait for 5 minutes and use another piece of cloth dampened with water to clean off residual hypochlorite.
5. The glass surface of the camera lens to be cleaned with a soft lens cleaning cloth.
6. The video display surface of LCD monitor should be wiped gently using a damp cloth and never spray cleaning solution directly on the display screen .

Specific Cleaning:

On Top and Bottom Shelf Surfaces

Wilsonart® Atlantis D525-60

(Blue inlaid laminate material)

Clean surfaces with warm water and mild soaps, such as those used for hands or dishes. Avoid cleansers containing abrasives, acids, or alkalis.

Stubborn stains may be removed with a 1.5 minute exposure to hypochlorite bleach (such as Clorox®) followed by a clean water rinse.

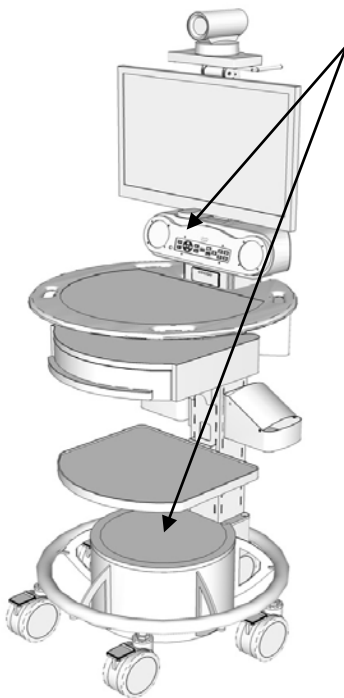
It is recommended that you not allow any of the following reagents to remain in contact with the decorative surface:

1. Hypochlorite bleach, except as described above
2. Hydrogen peroxide solution
3. Mineral acids, hydrochloric acid such as Lime-A-Way™, sulfuric or nitric acid
4. Caustic solutions containing greater than 2% lye, such as Drano®
5. Sodium bisulfate, such as Sani-Flush®
6. Potassium permanganate
7. Berry juices
8. Silver nitrate, in 1% concentration
9. Gentian violet
10. Mild silver protein, such as 20% Argyrol
11. Bluing
12. Fabric dye, such as Tintex® or Rit®
13. Alcohol containing 1% iodine in solution
14. **On Top and Bottom Shelf Surfaces**

DuPont™ Corian® Solid Surface

(Light-colored solid material)

1. General and everyday cleaning can be done by wiping the surface with a damp cloth or sponge, then dry with soft cloth or paper towels. An ammonia-based, non-abrasive cleaner may be used for stains.
2. Disinfect the surface with the appropriate sterilant or disinfectant.
3. Prolonged exposure to concentrated, undiluted disinfectant solutions may degloss or etch solid surface material and require refinishing to restore the surface to its original condition.
4. Using a wet Scotch- Brite® pad and bleaching and/or abrasive cleanser may restore the surface finish.

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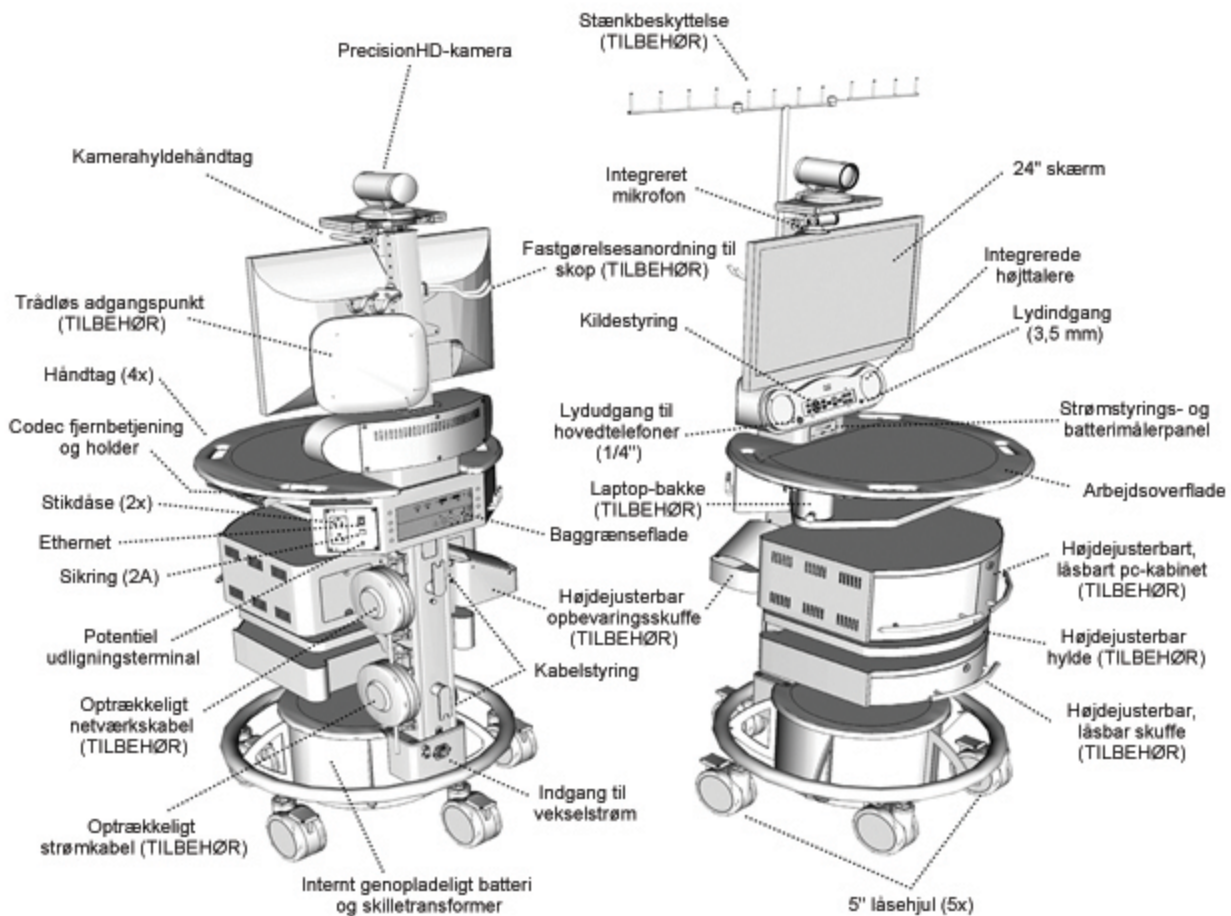
http://www.cisco.com/en/US/products/ps12152/tsd_products_support_series_home.html

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